

HYAVAGE™

VAGINAL PESSARIES

COMPOSITION

Each 2 g pessary contains 5.00 mg hyaluronic acid sodium salt, 1.00 mg tocopheryl acetate, 1.00 mg retinyl palmitate, semi-synthetic glycerides q.s.

CATEGORY

Medical Device.

PROPERTIES

HYAVAGE™ pessary is a preparation containing hyaluronic acid, the main component of the dermis' fundamental substance, the local use of which protects and improves the elasticity of the vaginal mucosa and at the same time creates the best possible environment for the migration and proliferation of the cells responsible for the regeneration process. Due to its hygroscopic properties, the hyaluronic acid forms a protective barrier from the external environment, thus favoring the tissue regeneration process. HYAVAGE™ pessaries contain low molecular weight hyaluronic acid that allows a high bioavailability and efficiency, accelerating the wound-healing mechanism of the vaginal mucosa. In addition, the combination of retinyl palmitate (vitamin A) and tocopheryl acetate (vitamin E) protects the vaginal environment against free radicals through their antioxidant activity.

CLINICAL INFORMATION

INDICATION

Treatment and prevention of vaginal disorders, including injuries, ectopia (known as folds) as well as other changes in the vaginal mucosa, also resulting from physical treatments (DTC, lasertherapy, cryotherapy, radiotherapy). Soothing and lubricant treatment, especially for post-menopausal patients.

CONTRAINDICATIONS

Known hypersensitivity to the components.

SIDE EFFECTS

There are no reported side effects to date. Inform your doctor and pharmacist about any side effect that is suspected to be caused by the use of the product.

EFFECTS ON ABILITY TO DRIVE AND USE MACHINES

There are no reported effects on the ability to drive or use machines.

METHOD OF ADMINISTRATION

1. Wash your hands thoroughly before opening the package. 2. Take a single pocket from the blister without damaging the remaining pockets. 3. Remove the pessary from the pocket. 4. Gently slide the pessary into the vagina trying as far as it go in order to prevent accidental product leakage.

DOSAGE

One pessary at night for 14 days consecutively. The treatment can be repeated as often as needed. In the event of vaginal conditions caused by physical treatments, the product should be administered since the first day of treatment.

WARNINGS

The use, especially if prolonged, of any product for topical use may give rise to sensitization phenomena; this being the case, stop the treatment and contact your doctor to start a suitable therapy. The treatment should be interrupted one day before in case of specific gynecological tests or treatments. The product must be administered exclusively by the vaginal route. There is no reported damage from overdose when used in normal conditions. Use during pregnancy should only be permitted when strictly necessary and under medical supervision. Do not use in case of allergy to any of the components. Use the product by the expiry date shown on package. Do not use the product if the package and/or the blister pocket are damaged. Keep out of reach of children. Do not swallow.

SPECIAL PRECAUTIONS FOR STORAGE

Store at room temperature between 0 and 25°C, away from light and heat.

PACKAGE

14 x 2 g pessaries.

This leaflet was last revised: 10/12/2019

CE 0373



 Lo.Li. Pharma S.r.l. - Via Sabatino Gianni, 14 - 00156 Roma - Italy
The local distributor is Medispharm Drugstore S.A.R.L - Lebanon