



Antibacterial Honey Tulle Dressing

Antibakterieller Honig-Tüllverband  
Pansement en tulle au miel antibactérien  
Miel Antibacteriana Apósito de Tul  
Gazeboinding med Antibakteriel Honning  
Antibakterielle lääkehuunajavälide  
Antibakteriell honungsmøslag av tyll  
Medicazione di tulle al miele antibatterico  
Antivakutriko Mélange d'Epiceau fôrás  
Antibakterielle Honing Wondverband  
Compresa de Gaze con Mel Bactericida  
Antibakteriell medový obvaz y zvlážd  
گاز پوشاندن عسل ضد باکتری  
Antibakteriel Ballı Tül Panşum  
ضمانة التولى الماء ضد المضاد للبكتيريا  
Antibakterijska obloga z gelom  
Gelkompress med antibakteriell honning  
Arkusz żelowy z miodem antybakteryjnym  
הארוך מיל אנטיבקטריאלי (Antibakteriel Honey) יונ'



It's not just honey – it's MEDIHONEY®

All of the products in the Derma Sciences MEDIHONEY® range contain MEDIHONEY® Antibacterial Honey, which is a standardised medical grade of antibacterial honey, such as Leptospermum scoparium (Manuka). This combination of Leptospermum honey has been put together using advanced scientific knowledge to maximise honey's potential to heal.

## Antibacterial Honey Tulle Dressing

MEDIHONEY® Antibacterial Honey Tulle Dressing (Tulle Dressing) is a sterile, non-adherent, non-absorbent wound dressing comprising of a strong woven dressing. Gewebe besteht, das mit MEDIHONEY® Antibakteriellem Honig getränkt ist.

## Antibacterial Honey Apinate™ Dressing

MEDIHONEY® Antibacterial Honey Apinate™ Dressing (Apinate™ Dressing) is a sterile, non-adherent wound dressing comprising of calcium alginate fibre impregnated with MEDIHONEY® Antibacterial Honey. The dressing will absorb wound exudate to form a soft gel.

## Antibacterial Honey Gel Sheet

MEDIHONEY® Antibacterial Honey Gel Sheet (Gel Sheet Dressing) is a sterile, non-adherent wound dressing comprising of MEDIHONEY® Antibacterial Honey and sodium alginate. These ingredients are combined using patented technology to create a honey dressing with a unique texture. The dressing will form a soft gel when in contact with wound exudate. Not recommended for dry wounds.

## Actions:

The MEDIHONEY® Tulle, Apinate™ and Gel Sheet Dressings are for use on acute and chronic wounds. Their actions include:

- assisting wound healing (providing a moist wound healing environment)
- protecting the wound by creating an antibacterial barrier against bacteria including antibiotic resistant strains
- fast, effective autolytic debridement to clean the wound, removing slough and debris
- rapidly removing malodour
- reducing the risk of infection from bacteria

## Indications:

	Antibacterial Honey Tulle Dressing	Antibacterial Honey Apinate™ Dressing	Antibacterial Honey Gel Sheet
	Antibakteriell Honig-Tüllverband	Antibakterielle Honig-Alginatverband	Gelkompress med Antibakteriell honning
Leg / foot ulcers	✓	✓	✓
Pressure ulcers	✓	✓	✓
Infected wounds	✓	✓	✓
Sloughy wounds	✓	✓	✓
Malodorous wounds	✓	✓	✓
Diabetic wounds	✓	✓	✓
Donor and recipient graft sites	✓	✓	✓
Burns	✓	✓	✓
Superficial wounds such as cuts, scratches, abrasions	✓	✓	✓
Oberflächliche Wunden wie Schnitte, Kratzer, Schürfwunden	✓	✓	✓
Surgical wounds	✓	✓	
Necrotic wounds	✓	✓	
Nekrotische Wunden	✓	✓	

Serious wounds should be managed under the supervision of a health care professional.

## Directions for use:

1. If required protect the wound edges with MEDIHONEY® Barrier Cream.
2. The Gel Sheet Dressing has protective liners on either side of the dressing. Remove these prior to use.

## Antibakterielle Ballı Aljinat Panşum

کریستکس اپینت میڈیهونی

## Antibakterijska obloga z apinato

Antibakteriell medový alginatbandage med honning

Antibakteriell alginatbandage med honning

Opatrunk Apinate™ z miudem antybakteryjnym

Apinate™ מיל אנטיבקטריאלי חומש

דב'אש אנטיבקטריאלי חומש

Antibacterial Honey Apinate™ Dressing

Antibakterieller Honig-Alginatverband

Pansement Alginate au Miel Antibactérien

Miel Antibacteriana Apósito de Alginato

Alginatebandage med Antibakteriel Honning

Antibakteerinen hunaja-alginaattisidos

Antibakteriell honung Alginatförband

Medicazione con alginate al miele antibatterico

Antivakutriko Mélange d'Epiceau

Antibakterielle Honing Wondverband

Penso di Alginato con Miel Bactericida

Antibakteriální medový alginátový obvaz

Antibakterijska obloga z alginatom

Antibakteriell honung Alginatbandage med honning

Antibakteriell alginatbandage med honning

Opatrunk Apinate™ z miudem antybakteryjnym

Apinate™ מיל אנטיבקטריאלי חומש

דב'אש אנטיבקטריאלי חומש

Antibacterial Honey Apinate™ Dressing Made in Canada.

Antibacterial Honey Tulle Dressing

product components

from New Zealand

packaged and sterilized in Canada

Antibacterial Honey Gel Sheet

Made in the USA.

Contact Us:

Derma Sciences, Inc.

104 Shorting Road

Toronto, Ontario M1S 5T4

USA Int: +1 609 514 4744

Canada Int: +1 416 299 4003

CE REP

Integra LifeSciences Services

Immeuble Sphère 2

97 Allée Alexandre Borodine

Parc Technologique de la Porte des Alpes

69800 Saint Priest - France

+33(0) 4 37 47 50 10

www.dermasciences.com

STERILE R

CE

1639

25°C

SAFETY

25°C

